





Firing recommendations for Dry milling

VITA ZYRCOMAT 6000MS (VITA)

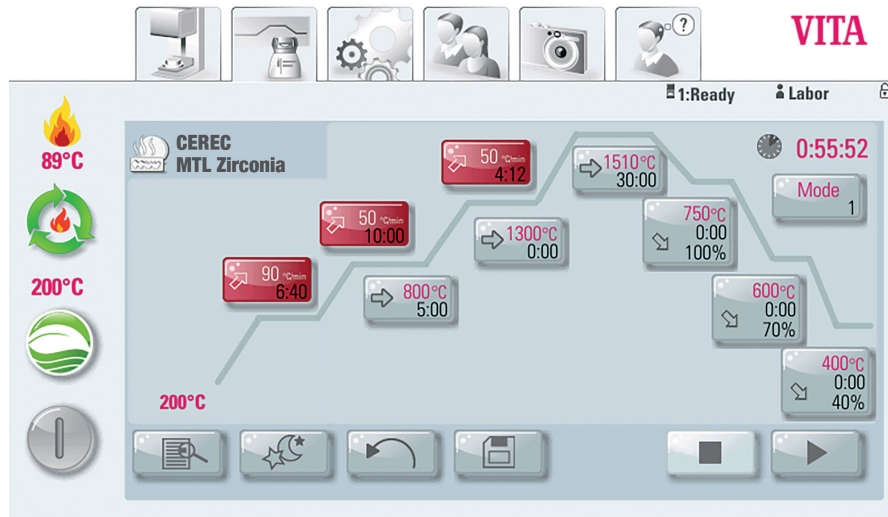

InLab ProFire (Dentsply Sirona)

#		 °C	
S1	99	800	5
S2	50	1510	30
S3	99	750	0
S4	120	50	0

Programat CS4 (Ivoclar Vivadent)

t_1	99	T_1	800	H_1	00:05
t_2	50	T_2	1500	H_2	00:00
t_3	15	T_3	1510	H_3	00:30
t_4	70	T_4	750	H_4	00:00
t_5	10	T_5	0	H_5	00:00
0	00:01				