

CEREC Premium SW 4.5.0 material pack description

This material pack requires a CEREC SW 4.5.0 software license, which must be activated via the license manager. You can obtain the “CEREC_Premium_4.5.0_MP_Setup.exe” material pack either as an automatic update or you can download the file manually from <https://my.cerec.com/en/products/cerec-premium-software.html>.

New material

- Manufactured by *Ivoclar Vivadent*, material: *Tetric CAD*, blocks I 12 and C 14; worldwide release
- Manufactured by *GC*, material: *CERASMART300*, blocks 12, 14 and 16; released only in Japan
- Manufactured by *Kuraray Noritake Dental*, material: *KATANA Zirconia* block (ST); worldwide release
- Manufactured by *Kuraray Noritake Dental*, material: *KATANA Zirconia* block (STML); worldwide release

Change of manufacturer


- *CELTRA DUO* material is listed as manufactured by *DENTSPLY SIRONA* (previously manufactured by *DENTSPLY*)

Name change

- A distinction is made between pre-integrated block sizes *Telio CAD A16 (S)* and *A16 (L)* (manufactured by *Ivoclar Vivadent*) by renaming the block versions in relevant processes to *Telio CAD Abutment*

Installation

- Go to <https://my.cerec.com/de-de.html> and download the file “CEREC_Premium_4.5.0_MP_Setup.exe”
- Double-click the downloaded file and select “Run”.
- Then accept the license agreement and select “Install”.

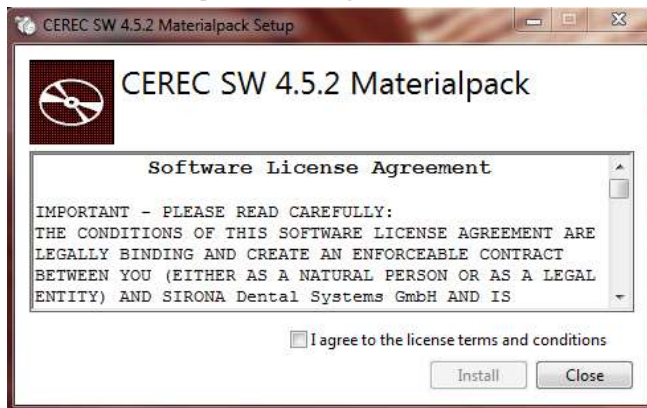

- The material pack will be installed automatically and the materials should be in the list the next time you start up your Premium software.
- You can check whether the block database is correct by clicking on the CEREC software control panel. The correct version of the “block DB” is 450.32.368

